
AT51032

INSTRUKCJA OBSŁUGI ŁAWKA NAWANNOWA
USER'S MANUAL BATH BENCH
NÁVOD K OBSLUZE SEDAČKA NA VANU
NÁVOD NA POUŽITIE SEDAČKA DO VANE
BEDIENUNGSANLEITUNG BADEWANNENSITZ
MANUEL D'UTILISATIONBANC DE BAIN
GEBRUIKERSHANDLEIDINGBADKUIPSTOEL
INSTRUCCIONES ASIENTO DE LA BAÑERA
MANUALE DI ISTRUZIONI SEDILE PER VASCA DA BAGNO
ANVÄNDARHANDBOK BADBÄNK

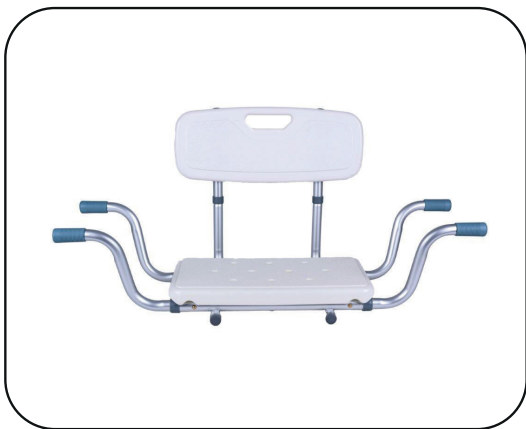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Thank you for purchasing this high quality medical device. Read this instruction manual carefully before use. Always follow the instructions and instructions for safe use. This instruction manual is intended for the patient as a user of the product.

Use

The bench facilitates the use of the bathtub by people with reduced mobility during physical hygiene activities.

Scope of delivery

- Bathtub seat, instructions for use.

The instruction manual is an integral part of the product and you should have it around.

- Product information, such as the model name, can be found on the product nameplate.

Indication

The use of the bathtub seat is indicated in case of:

- Movement instability and uncertainty
- Different problems with legs

Contraindication

The bathtub seat should not be used by people who are unable to control their balance while sitting

Application Notes

- Check the integrity of the bathtub seat before each use. If you notice any damage, immediately remove the product from use and contact your specialized distributor.
- Avoid direct sunlight and heat.
- It is not allowed to stand on the bathtub seat
- Inspect all parts for possible damage caused by transportation. If such damage is detected, DO NOT use the product.
- When using the bench and when adjusting the mechanisms, there is a risk that body parts of the user/user's companion get stuck or jammed between product particular parts. All activities should be carried out with extreme caution. Once the adjustment is complete, stabilize the position by carefully tightening the nuts/screws.
- There is a danger of slipping while sitting or getting up in the bath.
- Always enter into the bath with the help of assistants and seat down immediately on the seat.

Assembly instructions

- The assembly process can be carried out by a single person. In case of significant functional disorders, you may need help of a third party. Proceed in the following order:
- Insert the elements of the side frame into the seat frame one by one on both sides.
- When inserting the side frame element, lock it with the copper security pin.
- Then assembly the backrest in the same way.
- Check the accuracy of the assembled elements and the corresponding width of the bench. The frame of the bench should be equal to the width of the bath and the non-slip caps shall have full contact with the edge of the bath.

Sit on the bath seat, grab the edges of the bath, pick up and place your legs in the bath. Getting out of the device is done by grabbing the edges of the bath, stretching your legs, moving your leg closer to the edge out of the bath, the second leg, and turning your torso. The patient should not be taken from the bench inside the bath. Locate the bench so that it remains stable.

Features

- lightweight aluminum structure,
- two-stage width adjustment of 83-78 cm,
- non-slip caps to prevent the bench from moving,
- Seat with holes for easy drainage,
- Backrest for better user stability.

Cleaning and maintenance

1. Clean the frame with water and detergents (avoid corrosive substances).
2. Clean the seat and backrest with mild detergents.
3. Do not rub during cleaning.
4. The unused product should be stored in a dry room.
5. To maintain the operating parameters, it is extremely important to keep the product clean and store it properly.

ATTENTION:

Avoid more aggressive detergents such as alcohol, ammonia, solvents or hard brushes.

Technical data

Interior width 67 - 69 cm
Total width 83 - 78 cm
Depth 18 cm
Seat width 40 cm
Seat depth 23 cm
Height with backrest 40 cm Weight: 1.92 kg
Maximum user weight of 120 kg

THIS SIGN INDICATES THE MAXIMUM WEIGHT OF THE USER

METHOD OF DISPOSAL OF THE PRODUCT AFTER REMOVING FROM SERVICE

After the device is withdrawn from use, the medical device can be disposed of as normal household waste, except for electrical products - proceed in the manner appropriate for the disposal of electrical and electronic equipment.

WARNING:

In the event of a device-related "serious incident", which directly or indirectly led to, could have led to, or is likely to lead to any of the following events:

- a) death of a patient, user or another person, or
- b) temporary or permanent deterioration of the health of a patient, user or another person, or
- c) a serious risk to public health, such as a "serious incident" must be notified to the manufacturer and to the competent authority of the Member State, in which the user or patient resides. For Poland, the competent authority is the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products.

WARNING:

In the event of pain, allergic reactions or other alarming, unclear symptoms related to the use of the medical device, please consult a healthcare professional.

NOTE:

It is forbidden to use the product contrary to its intended use!

ATTENTION:

The manufacturer is not responsible for damage caused by neglect of maintenance, inadequate servicing or resulting from failure to comply with the recommendations contained in this manual.

NOTE:

If the product is used improperly, there may be a risk of tipping over. Please follow the sitting / standing up / moving instructions.

TARGET GROUP OF PATIENTS

Persons suffering from diseases, dysfunctions or injuries for the treatment/rehabilitation or compensation of which this device is intended (see section on intended use of the device in this instruction manual). The device can be purchased by the user on their own or on the recommendation of a doctor, therapist or other specialist. Whether you purchase the device yourself or on the recommendation of a doctor/therapist/other specialist, you must take into account the available sizes/necessary functions and variants of the device, the indications and contraindications for use and the information provided by the manufacturer.

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WARRANTY CARD

Model:
Serial number:

Point of sale stamp
and a legible signature of the Seller:

1. ANTAR Sp. J., 03-068 Warsaw, ul. Zawiańska 43, hereby grants a 12-month warranty on the rollator, starting from the date of purchase by the Buyer. Products financed by the National Health Fund are covered with the extended warranty period, which covers half of the useful life specified in the Regulation on Guaranteed Services, provided that the useful life is specified in the Regulation in months or years.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, send it immediately to ANTAR (as specified in point 1) or contact the point of sale.
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum permissible weight of the user as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts subject to normal wear and tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED. IN ORDER TO EXTEND WARRANTY PERIOD FOR PRODUCTS FINANCED BY THE NATIONAL HEALTH FUND, ATTACH A COPY OF THE ORDER FOR THE SUPPLY OF MEDICAL DEVICES BEING ORTHOPAEDIC ITEMS AND AIDS.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered.

ANTAR Sp. J.
03-068 Warsaw, ul. Zawiańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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