AT52335

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EN

Thank you for purchasing our product. Please read this user manual carefully before use. Do not operate the device until you have read and fully understood this manual. If you do not understand any warnings, notes, or instructions, contact a healthcare professional or the distributor to avoid product damage or personal injury. NOTE: Check all parts for damage that may have occurred during transportation. If any damage is found, DO NOT use the product. Contact your distributor for more information.

TARGET USER GROUP

This product is intended for individuals with illnesses, dysfunctions, or injuries that require treatment, rehabilitation, or functional compensation (see the section Intended Use in this manual).

The product can be purchased independently by the user or based on recommendations from a doctor, therapist, or other specialist. Whether purchased independently or recommended by a professional, one must consider the available sizes, necessary features and product variants, indications and contraindications, as well as information provided by the manufacturer.

INTENDED USE

This product is designed to assist people with physical disabilities or limited mobility, including those who are unable to walk. The wheelchair is intended for the transport of one person only.

INDICATIONS

Severe mobility limitations: Individuals with significant loss of muscle strength caused by neurological, muscular, or pain-related conditions that make independent movement impossible or very difficult.

Neurodegenerative diseases: Patients with conditions such as multiple sclerosis, Parkinson's disease, muscular dystrophy, or ALS, which cause progressive loss of muscle control and strength.

Fatigue: Individuals for whom long-distance walking or manual wheelchair propulsion is too physically demanding, leading to fatigue or exhaustion.

Age-related limitations: Elderly individuals with reduced strength or mobility may use an electric wheelchair to maintain independence.

Rehabilitation and recovery: Patients recovering from injuries or surgeries who are temporarily unable to walk can use the electric wheelchair as part of their rehabilitation process.

CONTRAINDICATIONS

Physical or mental impairments (e.g., visual impairment) that make safe use of the product impossible.

People who have difficulty judging distances or spatial awareness may not be able to maneuver an electric wheelchair safely.

Electric wheelchairs require a certain level of manual ability and coordination to operate the joystick or other control devices. Those who cannot operate these controls safely should not use the wheelchair.

Although many people with cognitive impairments can use electric wheelchairs safely, severe cognitive disorders may make safe operation impossible.

People unable to maintain a stable sitting posture may require additional support systems before using the wheelchair.

Epilepsy or other conditions that can cause sudden loss of consciousness or control may be contraindications.

Individuals with poor motor coordination or who cannot physically operate the controls safely should not use the wheelchair.

Users with impaired consciousness affecting their ability to drive safely should avoid use.

Severe heart or blood pressure conditions that may cause dangerous reactions under stress could prevent safe wheelchair use.

Advanced multiple sclerosis or other muscle control disorders may require customized wheelchair solutions.

Problems with depth perception, spatial awareness, or orientation can limit safe operation.

General malaise or recent surgeries, especially involving the spine or lower limbs, may require postponing wheelchair use until recovery.

PRECAUTIONS BEFORE USE

When getting on or off the wheelchair, do not step on the footrest—always place your feet firmly on the ground.

Warning: Never stand on the footrest when entering or exiting the wheelchair.

Practice operating the wheelchair in a safe, open area such as a park until you are comfortable.

Always fasten the safety belt before use.

Ensure all parts are securely fastened and the battery is sufficiently charged.

Maintain correct tire pressure (for pneumatic tires: <250 kPa). Incorrect pressure may cause instability, increase energy consumption, and reduce driving range.

PRECAUTIONS DURING USE

The maximum load of this wheelchair is 120 kg, and the maximum climbing angle is 10°.

Avoid steep slopes, uneven surfaces, high curbs, ditches, or driving sideways on inclines, as well as sharp turns or zigzagging.

Practice driving in a safe area and familiarize yourself with all wheelchair functions.

For your first drive, travel with an assistant and only in safe conditions.

Follow pedestrian traffic rules, not those for vehicles.

If assistance is required, always travel on sidewalks and pedestrian crossings, keeping to the right-hand side.

Avoid driving in the following conditions or areas:

Bad weather (rain, fog, strong wind, snow, etc.). If the wheelchair gets wet, wipe it dry immediately.

Poor or slippery roads (mud, snow, sand, gravel).

Roads with heavy traffic.

When there's a risk of front wheels getting stuck, move backwards instead (e.g., crossing railway tracks or boarding a subway car).

IMPORTANT: Always cross tracks or obstacles perpendicularly.

Coordinate movements with an assistant whenever possible.

When driving uphill or downhill, always move forward and use low speed when descending.

Avoid climbing high steps.

The wheelchair's maximum obstacle-climbing height is 6 cm, and the minimum approach distance is 50 cm.

Always approach obstacles perpendicularly.

If descending backward, ensure a caregiver is present.

Do not use the wheelchair to carry heavy loads or tow objects.

Do not stand or sit on the seat cushion incorrectly.

Electromagnetic interference:

Mobile phones, radios, or other transmitters may interfere with the wheelchair's electronics.

If strong electromagnetic interference occurs, switch off the interference source or immediately turn off the wheelchair power.

The wheelchair's electrical system is designed for user safety.

Any modifications without manufacturer authorization are strictly prohibited.

WARNING:

The user must not operate or remain seated in the wheelchair inside a motor vehicle, as sudden braking or an accident may cause injury.

When traveling by car, the user should transfer to a standard vehicle seat, fasten the seatbelt, and secure the wheelchair inside the vehicle.

NOTES FOR CAREGIVERS

Ensure the user's feet are securely placed on the footrests and that clothing does not touch the wheels.

When driving uphill or downhill, always keep the electromagnetic brake engaged and do not release the clutch.

Avoid sudden or harsh braking to prevent the user from being thrown from the wheelchair.

Ensure the clutch is engaged when the wheelchair is not in use.

Tips:

Environmental protection: Used batteries and wheelchairs must be disposed of in accordance with national environmental regulations.

Speed limit: In some countries, the maximum permitted speed for powered wheelchairs may be 6 km/h. When using the wheelchair, always comply with local road traffic regulations. Adjust your speed to the surrounding conditions and take extra care in areas with pedestrian traffic.

SAFE OPERATION INSTRUCTIONS

Operation on Different Surfaces

Climbing Slopes and Overcoming Obstacles: Approach steps or obstacles slowly, stopping at an appropriate angle. Position the front wheel close to the obstacle or step and rest it against the edge. Keep the front wheel perpendicular to the obstacle and maintain a forward driving direction. Continue moving slowly forward until the rear wheels pass over the obstacle. The maximum obstacle height that can be overcome is 4 cm.

Descending Slopes:

When descending a slope, travel at the lowest safe speed while maintaining full control of the wheelchair. Do not make sudden turns or brake abruptly. Do not drive downhill on slippery surfaces.

Ascending Slopes:

Keep your back straight while driving uphill. Do not lean your body beyond the wheelchair's backrest during ascent. Drive straight without zigzagging or making sharp turns. Avoid steep slopes — the recommended maximum incline is 10 degrees.

WARNING

It is strictly forbidden to use the wheelchair as a means of transporting goods or other people.

Driving on roads that violate traffic laws or national/local regulations is prohibited.

Use is forbidden for individuals with upper limb disabilities or difficulty operating the wheelchair.

Persons with mental disorders or lack of consciousness (unconscious or apathetic) must not use the wheelchair.

Do not drive on uneven roads.

Do not get on or off the wheelchair when the power switch is on and the parking brake is not engaged.

Avoid making sharp turns while driving at high speed.

Do not stand on the footrests.

Do not turn or maneuver on sloped surfaces.

Do not park on slopes.

Do not move transversely across an inclined surface.

Do not get in or out of the wheelchair on a slope.

Do not drive on wet roads or during rain.

Do not drive near slopes, bodies of water, or any areas without safety barriers.

Do not operate the wheelchair without properly using the seat belt and leg support straps.

Do not modify the wheelchair or replace its parts with non-original components without written consent from the manufacturer.

Do not lean out of the wheelchair while driving.

Do not place or hang clothes or objects on the back of the wheelchair near rotating parts.

Do not operate the wheelchair without first carefully reading this manual.

Do not store or leave the wheelchair near heat sources or heating devices.

Individuals with upper limb impairments, senile dementia, mental illness, or physical inability to function independently, as well as those advised by a doctor not to use a wheelchair, should not use it.

Before using the wheelchair, check the tire pressure, ensure that all accessories are properly installed, and confirm your familiarity with its operation.

When learning to drive the wheelchair, a caregiver must be present, and practice should take place in a safe environment.

Initial driving practice should be done in an open, safe area such as a park or square.

After the user is safely seated, fasten the seat belt.

For adjustable footrests, set the height to approximately 50 mm above the ground.

For wheelchairs without a reverse lock mechanism, do not lean backward or rest against the backrest during operation.

When encountering an obstacle, do not strike it directly with the wheelchair (especially important for elderly users with osteoporosis or high fracture risk).

When entering a roadway or pedestrian crossing, switch the wheelchair to manual mode, and the caregiver should push it safely.

When descending slopes, do so slowly, tilt the user's head and back as far backward as possible, and hold the handrails firmly.

When starting or braking an electric wheelchair, avoid sudden acceleration or braking.

The wheelchair's braking function is primarily intended for stabilization when driving on slopes, declines, or uneven terrain.

During driving, avoid sudden body movements — do not lean sharply left, right, forward, or backward.

Keep the wheelchair away from open flames to prevent ignition of the backrest, cushion, or seat belts.

When the wheelchair is not in use or during parking, turn off the controller and apply the parking brake.

Before use, check and confirm tire pressure once again.

The wheelchair is not suitable for driving on very slippery or uneven surfaces.

When transporting the wheelchair on a lifting platform, ensure that the anti-tip (swivel) wheel does not catch on other objects.

When using a lift or platform, the wheelchair controller must be switched off, and the handbrake applied.

During adjustment or assembly, exercise caution to avoid trapping fingers and prevent injury.

When testing the handbrake, ensure it functions properly and that the wheels are completely locked.

When moving the electric wheelchair, handle it carefully to avoid damaging the controller.

Do not exceed the maximum allowable incline when driving uphill.

Do not stop on inclined surfaces (uphill or downhill).

Do not attempt to climb obstacles higher than the specified maximum.

Do not cross ditches or obstacles beyond the allowed dimensions.

While operating the wheelchair, stay focused and avoid distractions.

The wheelchair must not be used to transport goods.

The wheelchair is designed for one person only.

The maximum load capacity of the wheelchair is 120 kg.

LIST OF COMPONENTS: Front Wheels, Rear Wheels, Frame, Controller, Motor, Seat with backrest, Battery box, User manual

TECHNICAL SPECIFICATION

Material:

Frame: carbon steel

Armrest: PU foam

Front wheel: solid PU

Rear wheel: pneumatic synthetic rubber tire, magnesium–aluminum alloy rim

Seat cushion: polyether sponge, polyester mesh fabric cover

Backrest cushion: polyether sponge, polyester mesh fabric cover

Joystick: iron, synthetic rubber, ABS plastic

Footplate: iron frame, plastic

Charger: plastic

Hand brake: iron, ABS plastic handle

Push handle: synthetic rubber

Anti-tipping wheel: metal, synthetic rubber

Protective boards: plastic

Battery box: plastic

Construction: foldable, mobile

Components: front and rear wheels, frame, controller, two electric motors, seat with backrest, battery box

Front wheels: 10 inches, solid (non-pneumatic)

Rear wheels: 16 inches, pneumatic (inflatable)

Seat: upholstered, with backrest and safety belt

Safety system: electronic brake (EABS), manual brake, anti-tip system, speed limiter, safety belt

Unfolded dimensions: 109 × 68 × 93 cm

Folded dimensions: 83 × 38 × 73.5 cm

Seat dimensions (width × length × thickness): 44 × 50 × 4 cm

Backrest height: 42 cm

Footrest height: 13 cm

Armrest height: 68 cm

Seat height from the floor: 48 cm

Net weight: 37 kg

Gross weight (with packaging): 40 kg

Battery weight: 2.65 kg

Drive system: dual motor drive

Motor: two brushed motors, 250 W each (2 × 250 W, DC 24V)

Controller: 360° joystick / manual control lever

Power supply: lithium-ion battery 24V / 10Ah

Charging time: 7–8 hours

Rated voltage: DC 24V

Maximum speed: up to 6 km/h

Theoretical driving range: over 20 km

Climbing ability: up to 6°

Turning radius: up to 120 cm

Maximum obstacle height: 4 cm

Maximum trench width: 10 cm

Braking distance on flat surface: ≤ 1.5 m

Braking distance on 6° slope: ≤ 3.6 m

Parking stability on slope: $\geq 9^\circ$
 Static stability (front, rear, side inclination): $\geq 9^\circ$
 Dynamic stability (front, rear, side inclination): $\geq 6^\circ$
 Power source: internal
 Protection class against electric shock: type B
 Protection degree against liquid ingress: ordinary type (non-sealed)
 Safety classification in the presence of anesthetic gases: Non-AP / Non-APG
 Operating mode: continuous
 Braking system: electronic EABS + manual brake
 Anti-tip system: yes
 Speed limiter: yes, up to 6 km/h
 Controller protection: yes
 Overheating protection system: yes
 Folding function: yes
 Maximum load: 120 kg

This symbol indicates the maximum user weight.

The listed dimensions may vary within ± 2 cm.

OPERATION

Unpacking, Assembly and Function Check

Unpack and check whether the wheelchair and its accessories are in good condition:

Using a utility knife, cut the sealing tape on the package carefully. Be cautious not to injure your hands or others nearby.

Remove all parts from the package and check that no components are missing or damaged. If you notice any missing or damaged parts, take photos and contact the seller as soon as possible.

Two people are required to lift the steel parts of the wheelchair. Grasp the steel frame on both sides, lift it simultaneously, and gently place it on the ground.

Quick Assembly and Use

Remove the wheelchair from the packaging: place it upright and press down firmly with both hands until the frame is fully unfolded.

Controller Installation:

Insert one end of the controller extension rod into the mounting tube located on the right side of the wheelchair frame.

Adjust the controller to the proper position and tighten the knob to secure it. The power cable should be routed straight and smoothly along the back of the wheelchair.

Footrest Installation:

Insert the footrest into the mounting socket on the frame and rotate it inward — a “click” sound indicates that the part is properly locked.

To remove the footrest, press the release button, pull it outward, and lift it upward.

Turning the Wheelchair On/Off:

Insert the two red plugs located at the rear and bottom of the wheelchair into the corresponding sockets, holding them firmly with both hands.

If the wheelchair will not be used for an extended period, fully charge the battery and then disconnect it from the power supply to ensure safety and prolong battery life.

Backrest Adjustment:

Hold the wheelchair's handle tube with both hands and lift it upward until the backrest is fully unfolded.

Make sure the spring latch (locking pin) is inserted into the hole at the lower end of the handle tube — a "click" sound will confirm proper locking.

To fold the backrest, press the button and perform the reverse action.

For safety, always check the locking mechanism after each adjustment.

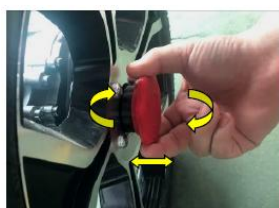
Using the Clutch:

When it is necessary to switch the wheelchair to manual mode, pull the clutch lever outward and position it outside the slot (this disengages the electric drive).

To return to electric mode, push the clutch back in so that the pin fits into the guide slot.

This operation should only be performed when there is an electric drive failure, battery discharge, or when the wheelchair must be pushed manually by a caregiver.

To activate manual mode, pull the clutch knobs on both sides of the wheels and rotate them until the stopper rests on the flat surface of the guide.

To activate electric mode, rotate the clutch knobs on both sides of the wheels so that the lock re-enters the slot.

The clutch will gently click into place when engaged.

Note:

Pay attention to the condition of the tires — when the clutch is disengaged (and the automatic electromagnetic brake is released), the wheels can rotate freely.

When switching from electric to manual mode, a caregiver's assistance is required to ensure user safety.

Using the Hand Brake:

Hold the hand brake lever and pull it upward.

When the brake pads make contact with the rear wheels, the wheelchair will stop moving.

Always prevent the wheelchair from slipping or coming into contact with other objects.

The rear wheels are pneumatic (air-filled) — maintain the air pressure according to the marking on the tire.

Avoid touching the wheels with your fingers while operating the brake to prevent pinching or injury.

Keep a safe distance from the wheels to avoid accidental trapping.

Securing the Backrest Strap:

Wrap the strap around the wheelchair frame and thread it through the buckle. Then wrap the backrest strap around the frame, pull it tight, and fasten it securely with the Velcro strip. Tip: Ensure that the tire pressure remains within 2.1 bar $\pm$ 10% and use a standard air supply.

When inflating, use a pressure gauge and do not exceed the recommended pressure — overinflation may cause the tire to burst or lead to personal injury.

CONTROLLER

1. Power Indicator: Each LED represents 20% of total battery capacity.
2. Power Switch: Turns the power on or off.
3. Horn: Activates the sound signal.
4. Speed Display: Shows current driving speed.
5. Speed Down Button: Decreases driving speed.
6. Speed Up Button: Increases driving speed.
7. Joystick: Controls driving direction, speed adjustment, and parking.

Turning on and Sleep Function of the Controller

Turning on the controller: Press the power button – the power indicator will light up sequentially from left to right and the buzzer will sound. This means the controller has been successfully activated.

Turning off the controller: Press the power button again – all LEDs on the panel will turn off, and the controller will stop sending signals. In emergency situations, the power button can also be used to immediately switch off the device.

Sleep mode: If the joystick is not used for 20 minutes, the controller will automatically power off and enter sleep mode. To restart the system, simply press the power button again.

In sleep mode, the current draw is ≤ 2.2 mA, which does not affect the battery.

Basic Controller Operation

Startup:

Ensure that the joystick is in the upright position (perpendicular to the ground).

Then press the power button – the power indicator should light up.

If the joystick is moved before activation, the safety system will block the wheelchair's movement and the indicator will flash.

Once the joystick is released, the wheelchair will resume normal operation.

Speed Adjustment:

The speed indicator shows the current speed relative to the maximum speed.

You can increase or decrease the speed using the "+" (increase) and "-" (decrease) buttons.

Each press changes the speed by 20%.

Holding the button allows smooth adjustment – release it when the desired speed is reached.

Speed is divided into 5 levels (1 light = 20% of maximum, 5 lights = full speed).

Directional Control (Joystick):

The joystick controls the direction and speed of the wheelchair depending on its tilt:

Joystick forward: move forward

Joystick backward: move backward

Joystick left: turn left

Joystick right: turn right

The more the joystick is tilted, the faster the wheelchair moves in that direction.

When starting or stopping, set the wheelchair to a low speed.

Once familiar with operation, increase to a safe and controlled speed.

Indoors, always use the lowest speed to avoid collisions.

In an emergency, release the joystick or move it in the opposite direction to stop immediately.

CHARGER AND BATTERY

The charger is used to charge the wheelchair. The charging socket is designed for use with the vehicle's charger.

A Y-shaped charging connector is used, allowing the battery to be charged directly.

During charging, the wheelchair's main power must be turned off, and the battery should be removed and placed in a safe, well-ventilated area.

The round ("Canon-type") charging connector can also be connected directly to the controller — the charging port is located beneath the joystick.

Charge the battery directly while it is installed in the wheelchair.
Charge the controller directly using the port located under the joystick.
The Y-shaped connector allows direct charging.
During charging, ensure that both the wheelchair power and battery are disconnected.
The actual charging time should not exceed 8 hours (recommended).
When using the round controller charging connector, make sure the controller is switched off.
The actual charging time should also not exceed 8 hours.

Charging Time

The wheelchair should be charged after continuous use of more than one hour or when the battery level drops below one-quarter (indicator turns orange).
If the wheelchair is used regularly, it is recommended to charge the battery daily.
If the wheelchair is not used for an extended period, charge the battery every 1–2 months to prevent degradation.

Charging Procedure

Step 1: Ensure that the charging socket is not blocked.
Step 2: Make sure the wheelchair's power switch is turned off.
Step 3: Insert the charger's output plug into the wheelchair charging port and ensure it is securely connected.
Step 4: Plug the charger's power cord into a wall socket or directly into the controller's connector.
During charging, the charger's indicator light glows orange, and the entire process takes approximately 7–8 hours.
Step 5: When the indicator turns green, the battery is fully charged.
Step 6: After charging, turn off the charger, then unplug it from the controller's socket.

Warning

Using the wheelchair without fully charging the battery may shorten its lifespan.
If the battery is not charged regularly, its capacity will decrease, reducing driving range.
The manufacturer is not responsible for reduced battery performance resulting from improper charging or use.

Tips

Do not expose the charger to high temperatures or place it near heat sources such as radiators or open flames.
During charging, ensure that the wheelchair's main power and the battery are disconnected.
Do not interrupt charging before completion — when the indicator turns green, charging is complete.
Do not continue charging after the green light appears to avoid overcharging.
Unplug the charger after charging — maximum charging time must not exceed 8 hours to prevent overcharging.
Charging time depends on ambient temperature — longer charging may be required in winter.

Safety Rules During Charging

Never modify or alter the charging circuit without the manufacturer's permission. The company is not responsible for accidents or damage caused by unauthorized modifications.
Do not disassemble or tamper with the charger.
Charge only in a well-ventilated area. Do not expose the charger to direct sunlight or humidity, and do not cover it with fabric or objects during use.
During operation, the charger may emit fan noise — this is normal and ensures proper cooling. The charger housing may become slightly warm, which is also normal.
The charger is not waterproof.
Do not place the charger on flammable materials such as oil, carpets, seats, or similar items.
Excessive discharge (below 1/5 of the remaining capacity) shortens the battery's service life.
During charging, keep away from sources of fire, as this may cause ignition or explosion of the battery.
Do not connect or disconnect the charging plug with wet hands or when the socket is damp — risk of electric shock.
Charger Specifications: Input voltage: AC 220 V Output voltage: DC 24 V Output current: 2 A During charging, the indicator glows orange and turns green when charging is complete. Follow the operating instructions provided with the charger.

Battery

Do not store the wheelchair or battery at temperatures below –10°C or above 50°C. Extreme temperatures may cause overheating, battery damage, or reduced lifespan. The wheelchair uses a maintenance-free battery — no replacement or electrolyte refilling is required. For additional battery-related information, refer to the warnings and labels on the battery itself.

Warning:

Do not open the battery cover or expose the battery to fire — risk of explosion or fire. Damaged batteries pose environmental and safety hazards. Dispose of them properly in accordance with environmental regulations or contact the dealer or manufacturer for safe disposal.

BATTERY CLEANING

If the battery becomes contaminated with water, battery acid, or dust, it may discharge quickly.
Follow the steps below to clean it properly: Turn off the power. Wipe the wheelchair housing around the battery with a clean, dry cloth. Wipe the battery with a clean cloth. If you notice white residue on the connector, remove it using a copper brush.

Tips:

Make sure the connector is properly installed. Do not use the wheelchair's battery to power other devices (e.g. communication or electronic equipment). The battery capacity may vary depending on ambient temperature — in winter, the driving range may be shorter.

BATTERY REPLACEMENT

Only use the same model or type of battery as recommended by the manufacturer.

If replacement is necessary for any reason, carefully read the instructions in this manual.

If you have any questions regarding battery replacement, contact the manufacturer or the dealer from whom you purchased the wheelchair.

Before replacing or installing the battery, always turn off the controller power.

Do not touch any metal objects with the battery connector to avoid the risk of short circuit or electric shock.

Read the warning label on the battery carefully before performing any actions.

Do not disassemble the battery to prevent electrolyte leakage and possible burns to the skin or eyes.

If the electrolyte comes into contact with your skin or clothing, rinse immediately with plenty of water.

If electrolyte gets into your eyes, rinse them with clean water for at least 15 minutes and seek medical attention immediately.

For safety, when handling the battery, wear rubber gloves, long-sleeved clothing, and protective goggles or a face shield.

If the battery casing or cover is cracked, deformed, or leaking electrolyte, replace the battery immediately.

SYSTEM DIAGNOSTICS

This product signals a malfunction in the electric wheelchair through a flashing LED on the controller and a repeating horn alarm sound.

When such a warning occurs, it indicates a failure in one of the wheelchair's components — such as the motor, brake, battery, wiring connections, or other internal parts.

The wheelchair is equipped with a built-in diagnostic function that allows you to read the fault code corresponding to the detected error.

This enables problem identification without additional service tools.

Audio signals:

Two short high-frequency sounds indicate the start of the alert sequence, followed by a series of beeps representing the error code.

If a continuous or looping alarm is emitted, it may indicate a serious malfunction.

In such cases, stop using the wheelchair immediately and eliminate the cause of the fault to ensure user safety.

Error Codes, Diagnosis, and Recommended Actions

Code 1

Problem: All LEDs are off

Diagnosis: Power off or sleep mode

Action: Turn on the power or check the fuse.

Code 2

Problem: Fewer LEDs indicate low battery level

Diagnosis: Low voltage / low battery power

Action: Charge the battery.

If the issue persists after charging, the battery may be damaged — replace it.

(Dispose of the battery properly according to environmental regulations.)

Code 3

Problem: Left drive wheel not working

Diagnosis: Left motor malfunction

Action: Check motor connections, wires, and connectors for looseness.

Code 4

Problem: Right drive wheel not working

Diagnosis: Right motor malfunction

Action: Check motor connections, wires, and connectors for looseness.

Code 5

Problem: Controller cable becomes unusually hot

Diagnosis: Current overload

Action: If the motor drive is jammed, use an ammeter to measure current and inspect the controller condition.

Code 6

Problem: No response when controlling direction

Diagnosis: Joystick malfunction

Action: Check joystick functionality and verify that its connection cable is secure.

Code 7

Problem: Controller unresponsive

Diagnosis: Controller failure

Action: Contact authorized service.

Code 8

Problem: Directional control error

Diagnosis: Signal error

Action: Contact authorized service.

Note:

The wheelchair's electrical system has been designed for safe operation.

Do not modify or adjust the electrical system independently, as this may cause damage or pose a safety risk.

WARNING:

In the event of a "serious incident" related to the product that has directly or indirectly caused, could have caused, or may cause any of the

following events:

- a) death of the patient, user, or another person;
- b) temporary or permanent deterioration of the health of the patient, user, or another person;
- c) serious risk to public health —
such a “serious incident” must be reported to the manufacturer and to the competent authority of the Member State in which the user or patient resides.

WARNING: If pain, allergic reactions, or any other concerning or unexplained symptoms occur while using the medical device, consult a healthcare professional.

WARNING: It is forbidden to use the product in any way other than as intended.

CLEANING AND MAINTENANCE

Use a damp cloth and mild detergent for cleaning. Do not allow electronic parts to come into contact with water. Do not use high-pressure cleaning equipment. Manual brake: Before each use, ensure that the wheels remain stationary when the brake is applied. Tire pressure: Check before each use. Cushions and backrests: Clean with warm water and mild soap if necessary. Controller: Always protect it from moisture and dirt. Fastening parts: Before each use, check that all screws are tight — tighten immediately if any are loose.

Daily Inspection

Before each use, check the following components. If any abnormalities are found, contact service or your dealer.

Controller: Can it be turned on and off normally? Are any joints or parts loose? Does the joystick operate smoothly in all directions?

Can the speed be adjusted? Does the power indicator work and show sufficient battery charge? Does the horn function properly?

Are there any unusual lights or sounds?

Motor: Are there any unusual noises? Does the electromagnetic brake function correctly (a clicking sound should be heard when starting and stopping)?

Clutch: Any unusual noises? Does the clutch operate properly?

Frame: Any unusual noises? Are any parts loose?

Tires: Are all parts stable? Are the tires damaged or underinflated? Any unusual noises?

Note: If you notice any irregularities, contact the manufacturer's service center or your dealer.

Regular Maintenance and Inspections

To keep the wheelchair in good technical condition, perform regular inspections.

Frequency of user inspections:

Weekly inspection:

Check whether the battery operates correctly. Check whether the front and rear wheels rotate smoothly without unusual noises. Check whether the lap belt is secure. Check whether the wheelchair frame is rigid and stable. Check whether moving parts are lubricated (with oil or grease).

Monthly inspection:

Check the screws and joystick near the armrests. Check the operation of the electromagnetic and manual brakes. Check the clutch. Check the joystick. Check the front and rear wheels. Check the tire tread depth. Check the electronic system connections and the charger plug.

Every 6 months (recommended):

Perform a detailed inspection of the wheelchair, including:

- motors and drive system (unusual noises, reduced performance)
- condition of the battery and battery compartment, charging socket and contacts
- electrical cables and connectors
- controller and safety functions
- brake operation and mechanical components

If any faults are detected and cannot be resolved independently, have the wheelchair inspected and repaired by a qualified service technician.

STORAGE: The product should be stored in a dry place, away from direct sunlight. Storage and transport temperature: -20°C to +55°C
Relative humidity: ≤93% The storage room should be clean, well-ventilated, and free of corrosive gases.

OPERATING ENVIRONMENT REQUIREMENTS: Ambient temperature: 20°C ± 15°C Relative humidity: 60% ± 35% Atmospheric pressure range: 860 hPa – 1060 hPa

DISPOSAL

This product contains electronic components. When it reaches the end of its service life, do not dispose of it with household waste.

To ensure proper disposal, deliver it to a designated collection point for electrical and electronic equipment.

Proper disposal helps prevent potential harm to the environment and human health, in accordance with applicable recycling regulations.

Used batteries must be disposed of in compliance with environmental protection laws.

PRODUCT LIFESPAN

The electric wheelchair is an active, non-sterile, reusable product that does not contain chemical substances.

There are no specific regulations defining its expiry period.

Its lifespan and safety depend on: product quality, usage habits, working environment, and regular maintenance.

Recommended service life: 2 years from the date of manufacture.

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Model:
Serial number:

Point of sale stamp

and a legible signature of the Seller:

1. ANTAR MEDICAL Sp. Zo.o., 03-068 Warsaw, ul. Zawisłańska 43, hereby grants a 12-month warranty on the product, starting from the date of purchase by the Buyer.
2. During the warranty period ANTAR undertakes to carry out the repairs necessary to restore proper functioning of the product, within 14 days from the date of receipt of the product by the service centre, together with the proof of purchase and the warranty card.
3. ANTAR reserves the right to replace the product if the repair is deemed uneconomic. Repair or replacement of faulty products does not extend the warranty period.
4. If the product is found to be malfunctioning, please contact your retailer/distributor or (if it is impossible) send it to ANTAR (as specified in point 1).
5. Before use, read the instructions carefully and follow them. If the product is used improperly and contrary to recommendations, the warranty will be irrevocably lost.
6. The warranty does not cover damage caused by external mechanical force, load exceeding the maximum allowed weight of the user, as described in the manual, as well as contact of the product with alcohol, greasy substances or petrol. The warranty does not cover parts which are subject to normal wear or tear during use, such as the bearing, rubber parts and others.
7. The product delivered to the service in connection with the complaint cannot cause an epidemic hazard.
8. THIS WARRANTY IS ONLY VALID TOGETHER WITH THE PROOF OF PURCHASE (BILL, RECEIPT, INVOICE). IN CASE OF A COMPLAINT, A PROOF OF PURCHASE MUST BE ATTACHED.
9. Complaints submitted without the proof of purchase and a warranty card with the product serial number will not be considered. 10. The warranty does not cover parts and components subject to natural wear and tear, such as wheels, tires, rubber pads, fuses, etc., the wear of which results from normal use of the product. The warranty period for batteries is 6 months.

ANTAR MEDICAL Sp. z o.o.
03-068 Warsaw, ul. Zawisłańska 43
Tel. 22 518 36 00, Fax 22 518 36 30
www.antar.net

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